



## UA-787AC Quick Response with EasyCuff™ Feature and AC Power Adapter



- Clinically validated for accuracy
- Time/date stamp
- Reminder alarms
- Irregular Heartbeat feature
- 30 reading memory
- Includes innovative easy to use cuff
- Comes with AC power adapter
- One button operation
- Latex free

Along with the EasyCuff and Irregular Heartbeat features, the UA-787AC includes time and date functionality and an AC power adapter.

The UA-787AC monitor allows users to keep track of the blood pressure more effectively by featuring a built-in 12 hour clock with alarm. The user can set up to three alarms as a reminder to take a reading. A time and date stamp also allows users to accurately track previous measurements with this blood pressure monitor.

Included with the blood pressure monitor is the *EasyCuff* feature, the latest innovation in cuff design from A&D Medical. With its unique semi-hard shell construction, the cuff readily conforms to the shape of your upper arm for a quick and proper fit.

This monitor includes a unique Irregular Heartbeat feature. The UA-787 monitor provides blood pressure and pulse rate measurements even when an Irregular Heartbeat occurs.

- Simultaneous readout of systolic/diastolic pressure and pulse rate
- Stores last 30 blood pressure and pulse rate measurements in memory with date and time stamp
- Set up to three reminder alarms
- *EasyCuff* size is two inches larger than the standard medium cuff to fit more arm sizes
- Extra-large digital display
- Large-print instruction guide with illustrations and quick start card in English, French and Spanish
- Four week blood pressure logbook included

For questions about *LifeSource* monitors, call us toll-free at 1-800-726-3364 and a specially trained representative will assist you, or visit us at [www.LifeSourceOnline.com](http://www.LifeSourceOnline.com).



# UA-787AC Quick Response with EasyCuff™ Feature

To obtain an accurate reading, it is important to use the correct cuff size. A cuff that is too small will result in a higher than expected reading and a cuff that is too large will result in a lower than expected reading. The **EasyCuff** feature has a larger cuff range (7.5 to 14.2" (19 - 36 cm)) than the size of the standard medium cuff (7.5 to 12.2" (19 - 31 cm)) to fit more arm sizes.

	PART NUMBER	UPC NUMBER	CUFF SIZE
UNITED STATES AND CANADA	UA-787AC	0-93764-60139-2	Medium 7.5 - 14.2" (19 - 36 cm)

Each unit comes with an AC power adapter, trilingual instruction manual, quick start card, four week blood pressure logbook and warranty card.

## Accessories

PART #	DESCRIPTION
UA-279	Small Blood Pressure Monitor Cuff: 5.1 - 7.9" (13 - 20 cm)
UA-280	Medium Blood Pressure Monitor Cuff: 7.5 - 12.2" (19 - 31 cm)
UA-281	Large Blood Pressure Monitor Cuff: 11.8 - 17.7" (30 - 45 cm)
UA-380	<b>EasyCuff</b> feature: 7.5 - 14.2" (19 - 36 cm)
UA-700CH	Cuff Hose
TB:181A	AC Power Adapter

## Technical Specifications

<b>Dimensions</b>	4.4" (l) x 6.4" (w) x 2.4" (h) (112 mm x 163 mm x 62 mm)
<b>Weight</b>	12.3 oz. (350 g) without batteries
<b>Display</b>	Digital: 20 mm character height Pressure and pulse displayed simultaneously
<b>Date/Time</b>	12 hour format (AM/PM), year (2003-2052), month, and day with automatic adjustment for leap years
<b>Memory</b>	30 readings
<b>Pressurization</b>	Automatic, using micropump
<b>Deflation</b>	Active exhaust valve
<b>Type</b>	Oscillometric
<b>Accuracy</b>	Pressure: ±3 mmHg or ±2%, whichever is greater; Pulse: ±5%
<b>Pressure Range</b>	20 mmHg to 280 mmHg; Pulse: 40 to 200 pulses/minute
<b>Power Source</b>	120V AC adapter (model TB:181A) or four type AA alkaline batteries (1.5 volt). Batteries not included.
<b>Battery Life</b>	Approx. four months with one daily measurement
<b>Operating Environ.</b>	50° to 104° F (10° to 40° C)
<b>Storage Environ.</b>	14° to 140° F (-10° to 60° C)

Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method within the limits prescribed by the American National Standards Institute (ANSI/AAMI SP10) for electronic sphygmomanometers. Manufactured by an ISO 9001 and ISO 13485 registered company. Engineered in Japan. Made in China.

## Packaging and Shipping Specifications

<b>Selling Unit (Box)</b>	<b>L x W x H (inches)</b>	<b>WT (lb.)</b>	<b>Individual Item Cube (cubic feet)</b>
UA-787AC	5 <sup>3/16</sup> x 5 <sup>7/8</sup> x 8	2.10	0.14
<b>Unit Case (10 units)</b>	<b>L x W x H (inches)</b>	<b>WT (lb.)</b>	<b>Case Cube (cubic feet)</b>
UA-787AC	29 <sup>1/2</sup> x 12 <sup>5/8</sup> x 11 <sup>5/8</sup>	25.00	2.51



1555 McCandless Drive  
Milpitas, CA 95035  
www.LifeSourceOnline.com

**LifeSource™**  
Your source for a lifetime of health



LifeSource and EasyCuff are trademarks of A&D Medical. ©2004 A&D Medical. All rights reserved.



Ask your A&D Sales Representative about other **LifeSource** products.

- Blood Pressure Monitors
- Heart Rate Monitors
- Nebulizers
- Digital Thermometers
- Personal Scales
- Stethoscopes

For questions about **LifeSource** monitors, call us toll-free at 1-800-726-3364 or visit our web site at [www.LifeSourceOnline.com](http://www.LifeSourceOnline.com)

LITLS787AC 05/04